## English

### Important

- Only use the thermometer in the way described in these directions for use.
- Do not hold the thermometer close to or in a source of heat.
- Do not clean with water that is hotter than 60 °C.

### General description (fig. 1)

- ➊ Temperature sensor
- ➋ Display
- ➌ On/off button
- ➍ Lid of battery compartment

### Body temperature

The normal body temperature varies between 36.5 and 37.5 °C. During an illness this temperature may go down (subnormal temperature), but it usually goes up ('fever').

The most reliable way to take a person's temperature is rectally. Measuring beneath the tongue produces values that are slightly below the actual body temperature (the normal temperature beneath the tongue is 36 to 36.8 °C). Measurement in the armpits produces unreliable values.

### Taking someone's temperature

- Press the on/off button. The test value of 42 °C is displayed for a few seconds and the °C symbol starts to blink.
- Put the temperature sensor beneath the tongue (fig. 2) and keep the mouth closed, or:
- Insert the temperature sensor rectally (1 cm deep).
- After approximately 1 minute you will hear a beep and the °C symbol will stop blinking. This means that the measurement has been completed.
- Read the temperature on the display.
- Press the on/off button to switch the thermometer off.
- If you do not switch it off, the appliance will switch itself off automatically after approx. 20 minutes.

### Error message after 'measurement completed' signal

- The dashes appear and the °C symbol starts blinking (fig. 3):
  - the temperature measured is lower than 32 °C.
- Blinking °C symbol (fig. 4):
  - the temperature measured is higher than 42 °C.

### Memory

If you keep the on/off button pressed, the last temperature measured will be displayed after three seconds.

### Cleaning

Clean the appliance with water that is not hotter than 60 °C or with medicinal alcohol.

*For hospitals:* immerse in water containing the prescribed disinfectant (see fig. 7) for 1 hour at the most.

### Replacing the battery

When the symbol shown in fig. 5 appears on the display, the battery (type LR41) must be replaced (see fig. 6). Hand in the empty battery at a chemical waste collection point.

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